

CARTIMAX PREPARATION GUIDE



1 Remove the 3 components from box.



2 Using standard aseptic/sterile technique, place into the sterile field.



3 Thaw fibers in warm sterile irrigant for approximately 20 minutes. Note: Decant per procedure immediately after thawing



4 Immediately after thawing the Fibers, put the strainer cap on the jar. Drain the fluid from the Fibers.



5 Fill jar with sterile irrigant (saline or 5% Dextrose in Lactated Ringer's Solution).



6 Decant the sterile irrigant following the instruction from step 4.



7 Open the CAM and pour it into the jar of Fibers.



8 Using the spatula, mix the CAM and Fibers together. CartiMax is now ready to use.